LEAFLET

# Battery Pack SPL3021

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**WARNING**

To avoid injury, always read this *Instructions for Use* and accompanied documents before using the product.

**Mandatory to read the *Instructions for Use*.**

Design Policy and Copyright

© and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

## Battery Pack

The battery life is variable (2-5 years) and mainly depends on proper charging practices.

Recharge the batteries on a regular basis (minimum of once every month) to prolong their life span. Avoid the batteries from reaching a low charge state.

Remove the battery pack from the lift when it is not used for a long period.

Charge at least 8 hours, until battery charger shows green light.

**CAUTION**

**For battery charging instructions always refer to the specific operating instructions for your lifter and any separate Arjo battery charging instructions (if applicable).**

**CAUTION**

**Refer to your local regulation for recycling and disposal of battery packs.**

## Symbols

| General Symbol  | Key to Symbol  |
|---|--|
|  | This symbol points out the manufacturer's name and address. May also point out manufacturing date. |
|  | This symbol points out the manufacturer's catalogue number.  |
|  | This symbol points out the manufacturer's serial number.   |
|  | "Separate collection" for all batteries and accumulators as per the WEEE Directive.                |
|  | Consult Your Products Instruction for Use.   |

## Technical Data

|  |   |
|--|---|
| Battery pack weight  | 4.9 kg (10.8 lbs)                                 |
| Battery pack type and part number (rechargeable lead-acid) | SPL3021   |
| Battery pack capacity                                      | 6 Ah  |
| Operating temperature / humidity range                     | 5 °C to 40 °C (41 °F to 104 °F) / 25-85% RH max   |
| Storage temperature / humidity range                       | -25 °C to 70 °C (-13 °F to 158 °F) / 0-93% RH max |

## Allowed Combinations

|                       |   |
|-----------------------|---|
| Arjo Battery          | SPL3021   |
| Arjo Battery Chargers | SPL3024<br>KPA2000-AU<br>KPA2000-CH<br>KPA2000-EU<br>KPA2000-GB<br>KPA2000-US |

### CAUTION

**DO NOT USE BATTERY CHARGER KPA0101 for charging battery pack SPL3021. These items are not compatible. Doing so may lead to unwanted situations (e.g. battery heating, overheating).**

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**HONG KONG**

Arjo Hong Kong Limited  
Room 411-414, 4/F, Manhattan Centre,  
8 Kwai Cheong Road, Kwai Chung, N.T.,  
HONG KONG  
Tel: +852 2960 7600  
Fax: +852 2960 1711

**ITALIA**

Arjo Italia S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjo.com

**MIDDLE EAST**

Arjo Middle East FZ-LLC  
Office 908, 9th Floor,  
HQ Building, North Tower,  
Dubai Science Park,  
Dubai  
Al Barsha South  
P.O Box 11488, Dubai,  
United Arab Emirates  
Direct +971 487 48053  
Fax +971 487 48072  
Email: Info.ME@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**

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Arjo Inc.