MGH00144

Valid from / en cours de validité depuis le / en espera de validación desde el / in corso di validità dal / gültig seit dem

16/11/2021

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PRODUCT SHEET

valid from 16/11/2021

<b>IDENTIFICATION OF THE MEDICAL DEVICE</b>	Type	Rechargeable battery	
	Commercial designation	Batterie medical DRAGER JM103 2,4V 800mAh FC	
	Reference	MGH00144	
	Basic UDI	3660766RechargeablePC	
	EAN	3660766502391	
	Brand	NX	
	Compatible / Original battery	Compatible	
	Packaging	Unitary	

RECOMMENDED USAGE

Follow the instructions and recommendations specific to each model, using the technical instructions and document resources from the devices in which the battery is used

Brands	Equipment	Models	PN
Dräger / Siemens / Minolta vickers	Bilirubinometre	JM103	JM103/2HR-4UC

<b>Identification</b>	<b>GENERAL TECHNICAL CHARACTERISTICS</b>	<b>Chemistry</b>	NiMh
		<b>Type</b>	AAA
		<b>IEC designation</b>	2 HRM12/43
		<b>Rated voltage</b>	2,4V
		<b>Nominal capacity</b>	0,8Ah
		<b>Internal resistance <math>\Omega</math></b>	120m $\Omega$

The voltage and the actual capacity in use can be affected by several factors, especially the temperature, the discharge current, the pack's history (ex:use, storage), etc

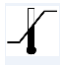
<b>ELECTRICAL CHARACTERISTICS</b>	<b>CHARGE</b>	Maximum charging voltage	3,4V
		Standard charging current (15h)	80mA
		Fast charging current (2,5h)	600mA
	<b>DISCHARGE</b>	Range of operating voltage	2V at 3,4V
		Min tension in discharge	2V
		Max discharge current	800mA
		Lifespan 80% DOD (0,5 C)	500 Cycles
	<b>CONTROL ELECTRONICS</b>	Electrical protection	No
		Over voltage detection threshold	No
		Low voltage detection threshold	No
Over current detection threshold		No	

These devices not only designed to protect the pack in case of an equipment failure. They must not be used to control the discharge. The protection circuits have a response time of a few milliseconds.

<b>MECHANICAL CHARACTERISTICS</b>	<b>Dimensions (+/- 2mm)</b>	Length	20,5mm
		Width	11mm
		Depth	45,3mm
	Weight (+/- 5g)	26,2Gr.	
	Mechanical protection	Shrink Sleeve	
	Wire length (+/- 10mm)	120mm	
Terminal	CL; Molex; 2 Wy; 51005; male housing; 2.0mm		

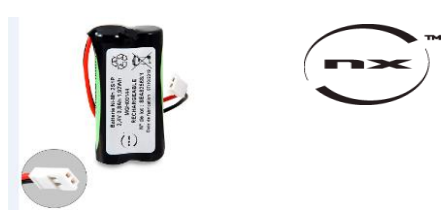
<b>CONDITIONS OF USE, STORAGE, AND TRANSPORT</b>	<b>CONDITIONS OF USE</b>	Charging temperature	-10°C at +35°C
		Discharge temperature	-15°C at +55°C
	<b>CONDITIONS OF STORAGE</b>	Storage temperature	-5°C at +35°C
		Humidity range	30% - 95%
		Frequency of maintenance charge at 20°C	2 Months
	<b>TRANSPORT</b>	Max storage time	2 Years
		UN code	Class 9
		ADR/RID classification	Class 9
		IMDG classification	Class 9
		IATA classification	Class 9

<b>INSTRUCTIONS</b>	<b>COMMISSIONING</b>	<ul style="list-style-type: none"> <li>• Check the batteries and the connectors: wires not damaged, battery not swollen, burnt smell, oxidation of the connectors, leak...</li> <li>• Respect the polarity</li> <li>• Do a full charge with the adequate charger before the first use</li> </ul>
	<b>CHARGE</b>	<ul style="list-style-type: none"> <li>• Use an adequate charger</li> <li>• The battery is warmer during the charge: during the first charge, check that the battery's temperature stays in the temperature operating ranges.</li> <li>• In case of an abnormal heating, stop the charge by unplugging the charger within the realms of possibility, remove the battery from the equipment de l'équipement, have the equipment, the battery and the charger checked by a technician.</li> </ul>
	<b>CASE OF NON-WATERPROOF BATTERIES</b>	<ul style="list-style-type: none"> <li>• It is normal to observe a release of gas during the charge and use. Do not smoke. Place in suitable premises.</li> <li>• Open batteries need regular maintenance carried out by a qualified technician.</li> </ul>
	<b>CASE OF LITHIUM ION BATTERIES</b>	There is a fire hazard with lithium ion batteries in the following cases: overload, short circuit, charge and use outside the voltage and temperature ranges.
	<b>WARNINGS</b>	<ul style="list-style-type: none"> <li>• Read the instructions of your device.</li> <li>• Only use in compatible devices.</li> <li>• Respect the load and storage conditions.</li> <li>• Do not use if the battery is damaged, do not burn, do not pierce, do not dismantle or modify. The protection circuits protect the battery and the equipment: do not deactivate them.</li> </ul>

<b>EXPLANATION OF SYMBOLS</b>	<b>LABELLING</b>	
		Catalogue reference
		Lot number
		Manufacturer's address
		To recycle in a suitable salvage and recycling structure
		Read the product sheet and the instruction manual
		Production date
		Fragile, handle with care . Can be broken or damaged if not handled carefully
		Keep away from sunlight. "Keep away from heat".
		Keep dry. Needs to be protected from moisture. "Keep away from rain"
		Temperature limit. Indicates the temperature limits to which the medical device can be safely exposed. The upper and lower limits of temperature are indicated adjacent to the upper and lower horizontal lines.
		Humidity limitation. Indicates the range of humidity to which the medical device can be safely exposed. The humidity limitation are indicated adjacent to the upper and lower horizontal lines.
		Unique device identifier. Indicates a carrier that contains unique device identifier information NOTE This symbol identifies the UDI carrier, including the AIDC (Automatic Identification and Data Collection) and human readable information.

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