MMP28122

Valid from / en cours de validité depuis le / en espera de validación desde el / in corso di validità dal / gültig seit dem / válida a partir de

01/08/2024

Product Sheet (EN)	2
Fiche Produit (FR)	4
Ficha producto (ES)	6
Scheda Prodotto (IT)	8
Eigenschaften (DE)	10
Ficha do Produto (PT)	12

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PRODUCT SHEET

valid from 01/08/2024

IDENTIFICATION OF THE BATTERY FOR MEDICAL DEVICE	Type	Rechargeable battery	 
	Commercial designation	Batterie médicale rechargeable 2x Nx 1.2-12 2S1P ST1 24V 1.2Ah Molex	
	Reference	MMP28122	
	Basic UDI	3660766RechargeablePC	
	EAN	3660766658883	
	Brand	NX, a brand of Upergy	
	Compatible / Original battery	Compatible	
	Packaging	Unitary	

RECOMMENDED USAGE

Follow the instructions and recommendations specific to each model, using the technical instructions and document resources from the devices in which the battery is used

Brands	Equipment	Models	PN
Arjo	bath	M4/5 Malibu hoist	

IDENTIFICATION	GENERAL TECHNICAL CHARACTERISTICS	Chemistry	Lead-acid
		Type	AGM
		IEC designation	2 AGM97/43/52-1
		Rated voltage	24V
		Nominal capacity	1,2Ah
		Internal resistance Ω	<290mΩ

The voltage and the actual capacity in use can be affected by several factors, especially the temperature, the discharge current, the pack's history (ex:use, storage), etc

ELECTRICAL CHARACTERISTICS	CHARGE	Maximum charging voltage	29,4V
		Standard charging current (15h)	120mA
		Fast charging current (2,5h)	1200mA
	DISCHARGE	Range of operating voltage	21V to 29,4V
		Min tension in discharge	21V
		Max discharge current	18A
		Lifespan 80% DOD (0,5 C)	> 500 Cycles
	CONTROL ELECTRONICS	Electrical protection	Yes
		Low voltage detection threshold	No
		Over voltage detection threshold	No
		Over current detection threshold	Yes

These devices not only designed to protect the pack in case of an equipment failure. They must not be used to control the discharge. The protection circuits have a response time of a few milliseconds.

MECHANICAL CHARACTERISTICS	Dimensions (+/- 2mm)	Length	198mm
		Width	44mm
		Depth	54mm
	Weight (+/- 5g)	1,1kg	
	Mechanical protection	Shrink sleeve	
	Wire length (+/- 10mm)	1300mm	
	Terminal	Molex 2 ways	

CONDITIONS OF USE, STORAGE, AND TRANSPORT	CONDITIONS OF USE	Charging temperature	10 to +40°C
		Discharge temperature	-15 to +50°C
	CONDITIONS OF STORAGE	Storage temperature	-15 to +40°C
		Humidity range	15% - 80%
		Frequency of maintenance charge at 20°C	6 Months
		Max storage time	2 Years
	TRANSPORT	UN code	2800
		ADR/RID classification	Not regulated
		IMDG classification	Not regulated
		IATA classification	Exempt
		Power capacity	28,8 Wh

INSTRUCTIONS	COMMISSIONING	<ul style="list-style-type: none"> • Check the batteries and the connectors: wires not damaged, battery not swollen, burnt smell, oxidation of the connectors, leak... • Respect the polarity • Do a full charge with the adequate charger before the first use
	CHARGE	<ul style="list-style-type: none"> • Use an adequate charger • The battery is warmer during the charge: during the first charge, check that the battery's temperature stays in the temperature operating ranges. • In case of an abnormal heating, stop the charge by unplugging the charger within the realms of possibility, remove the battery from the equipment de l'équipement, have the equipment, the battery and the charger checked by a technician.
	CASE OF NON-WATERPROOF BATTERIES	<ul style="list-style-type: none"> • It is normal to observe a release of gas during the charge and use. Do not smoke. Place in suitable premises. • Open batteries need regular maintenance carried out by a qualified technician.
	CASE OF LITHIUM ION BATTERIES	There is a fire hazard with lithium ion batteries in the following cases: overload, short circuit, charge and use outside the voltage and temperature ranges.
	WARNINGS	<ul style="list-style-type: none"> • Read the instructions of your device. • Only use in compatible devices. • Respect the load and storage conditions. • Do not use if the battery is damaged, do not burn, do not pierce, do not dismantle or modify. The protection circuits protect the battery and the equipment: do not deactivate them.

EXPLANATION OF SYMBOLS	LABELLING	
		Catalogue reference
		Lot number
		Manufacturer's address
		To recycle in a suitable salvage and recycling structure
		Read the product sheet and the instruction manual
		Production date
		Fragile, handle with care . Can be broken or damaged if not handled carefully
		Keep away from sunlight. "Keep away from heat".
		Keep dry. Needs to be protected from moisture. "Keep away from rain"
		Temperature limit. Indicates the temperature limits to which the medical device can be safely exposed. The upper and lower limits of temperature are indicated adjacent to the upper and lower horizontal lines.
		Humidity limitation. Indicates the range of humidity to which the medical device can be safely exposed. The humidity limitation are indicated adjacent to the upper and lower horizontal lines.
		EU Battery Regulation 2023/1542
		Unique device identifier. Indicates a carrier that contains unique device identifier information NOTE This symbol identifies the UDI carrier, including the AIDC (Automatic Identification and Data Collection) and human readable information.

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